


Thopaz™ Quick Card




This Quick Card in no way replaces the Instructions for Use REF 200.0685



Prepare Thopaz for use

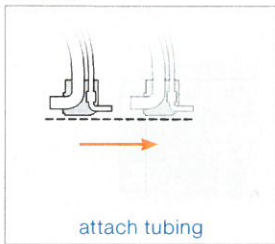


Turn Thopaz on

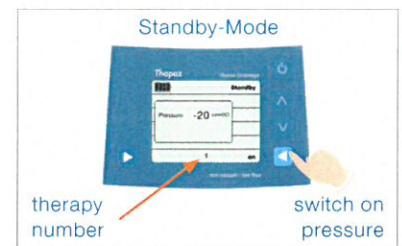



Perform functional check

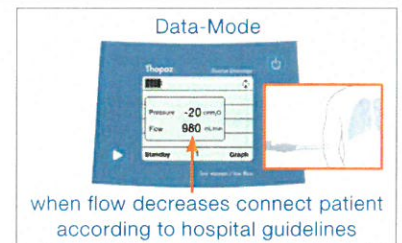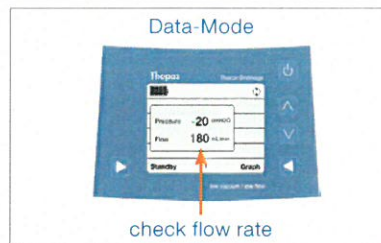



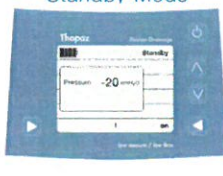
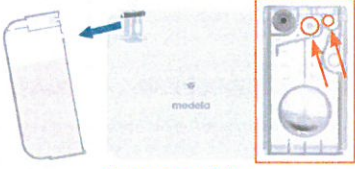
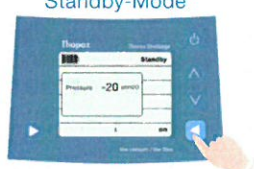
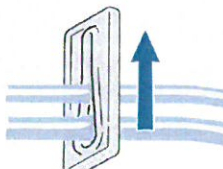
Adjust pressure (when pump is running)

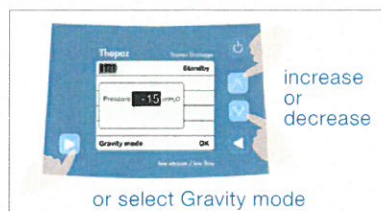


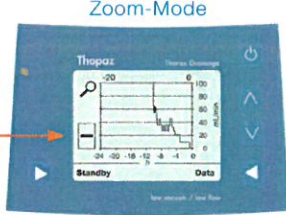
Check therapy progress

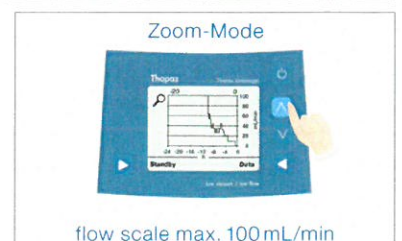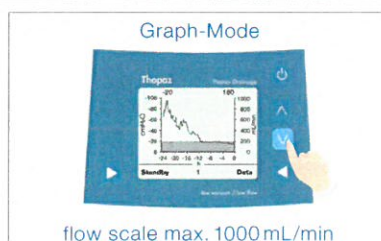

Change canister

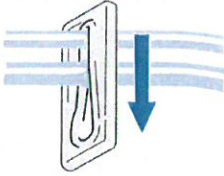 <p>clamp off tubing</p>	<p>Data-Mode</p>  <p>press longer than 3 seconds</p>	<p>Standby-Mode</p>  <p>Thopaz in Standby-Mode</p>
 <p>change canister seal canister and dispose</p>	<p>Standby-Mode</p>  <p>leave standby-mode</p>	 <p>release clamp</p>



Catheter check

<p>catheter open</p> <p>catheter clogged</p> 	<p>Zoom-Mode</p> 	<ul style="list-style-type: none"> - Only visible in Zoom-Mode when flow is 0 mL/min - Patient must inhale and exhale deeply and slowly
---	---	---

Alarm management

<p>BEEP</p>  <p>error number</p>	<p>BEEP</p>  <p>press simultaneously</p>	 <p>follow advice on screen</p>
---	---	--

Switch off Thopaz
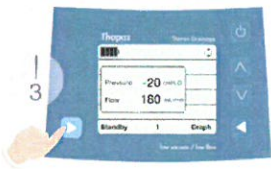
<p>Switch to Standby</p>  <p>press longer than 3 seconds</p>	<p>Switch off</p>  <p>press once quickly</p>
---	---


Medela AG
 Lättichstrasse 4b
 6341 Baar, Switzerland
www.medela.com

International Sales
 Medela AG
 Lättichstrasse 4b
 6341 Baar, Switzerland
 Phone +41 41 562 51 51
 Fax +41 41 562 51 00
ism@medela.ch
www.medela.com

CAUTION:
 Federal US law restricts this device to sale by or on the order of a physician.