

SIEMENS CT - Skift til lejetop max 300kg

4.3.5 Changing the MPT table top

The optional table top of the multi purpose table can be changed according to your clinical needs.



For a better and safer handling use two people to move the table tops.

Exchanging a table top

When exchanging table tops on the multi purpose table (MPT), you proceed as follows:


- 1 Remove the table top from the MPT.
- 2 Take care of your hands.
- 3 Place the table top in the wall holder vertically, with the heavy side down.

The illumination on the gantry operator panel, and on the control box remains.



You cannot move the table without a table top.

- 4 Place a new table top on the MPT.
- 5 Press down the table top until it locks in place.

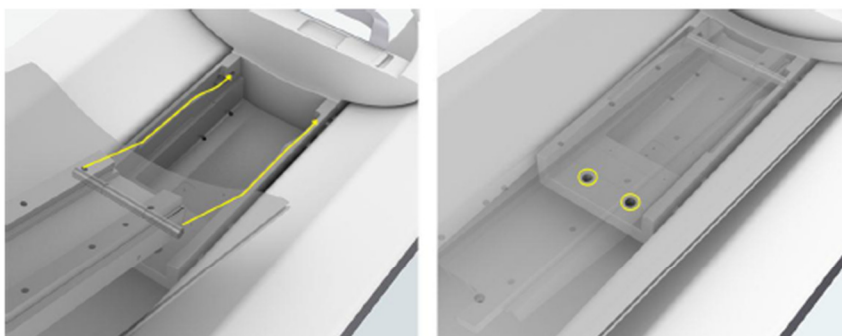

After exchanging the table top

The illumination on the gantry operator panel, and on the control box remains.

- ◆ Press any key on the control box, or on the gantry operator panel.