

4.5.2. *Cooling Therapy Control Panel* After completion of the Self Test, the Cooling Therapy control panel appears (see Figure 4-12).

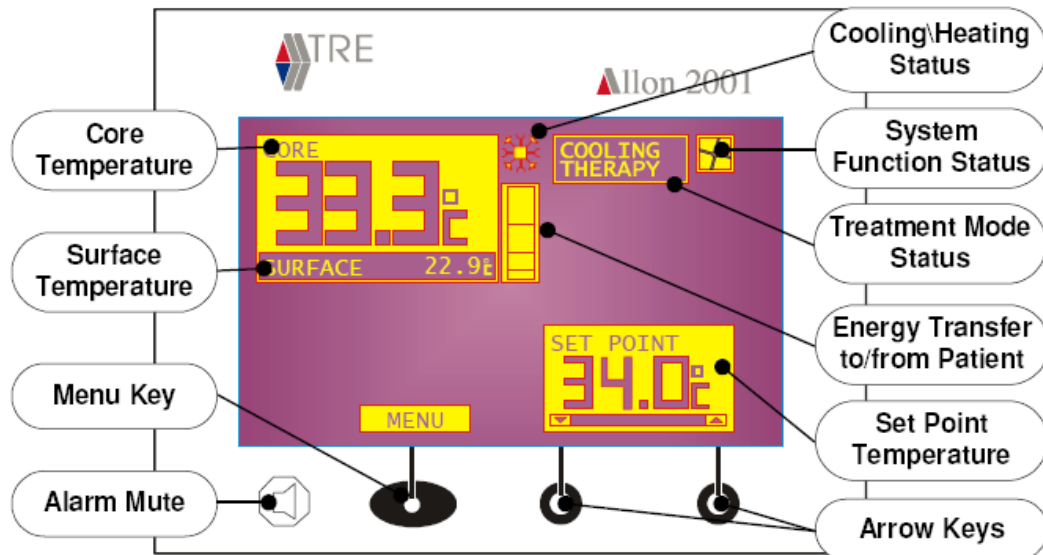


Figure 4-12 Cooling Therapy Control Panel

Table 4-2 Key to Control Panel Symbols

Symbol	Description	Explanation
	Heat transfer to / from patient	Graphical representation of the intensity of heat transferred between the patient and the ALLON system
	Heating/Cooling Status	Indicates the direction of the heat transfer. <ul style="list-style-type: none"> • Snowflake icon - Heat transferred from the patient • Sun icon - Heat transferred to the patient
	System Function 'Status	Indicates the condition of the pump. If the icon is not displayed, the pump is not functioning and water does not flow to the ThermoWrap.